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Smart Bracelet User Manual

Please check the instructions before use

Thank you for purchasing and using CB601 smart heart. Please read this manual to have a comprehensive understanding and simple operations. This bracelet with heart rate and blood pressure function, which realizes data transmission with the use of "Bluetooth 4.0" protocol. In addition, it has time, week, steps, calorie consumption, exercise distance calculation, sleep monitoring, call, message, alarm clock reminders, camera and other functions, and the user can synchronize the data with the phone for reference, detection and analysis. Goods in kind prevail, which are subject to changes without notice!

1. The packaging list

Bracelet*1 Charging clip*1 Box*1 User instructions*1

2. The connection guide


1. Search keyword "H Band" in the Android market, Apple store to download it for installation.
2. Or scan the QR Code to download H Band:

H Band

3. Bluetooth connection

- Step 1: Turn on the Bluetooth of the phone;
- Step 2: Open the "H Band" APP application interface, and click on the connected device in the "settings" menu;
- Step 3: APP searches the name of the Bluetooth bracelet, and click the name of the Bluetooth device such as: "CB601 Pro". It goes back to the setting interface automatically after the click, and pop-up Bluetooth matching request interface. Click on the "match" to make connection; operations in the following figure: if no device is found, pull-down to refresh the page manually.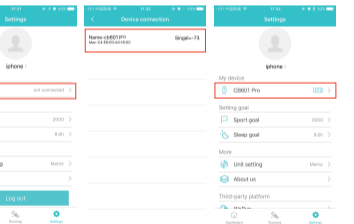

4. Charging instructions

Please ensure that the battery charge is sufficient at the first use. If the power is too low to be started up, please use the

dedicated point-contact charging clip to charge the bracelet. (As shown in the figure)

Charging mode: clip-type charging interface
Charging voltage: DC5V
Charging current: 200~500mAh
Power supply type: 100mAh polymer lithium battery
Bluetooth standard: Bluetooth 4.0 BLE data transfer

5. Bracelet display interface

After the device is powered on, the time interface is displayed by default. Press the touch key. Switch between different function interfaces.

Main interface Steps Count Distance Blood Pressure

Tips: Some interfaces are turned off by default. If necessary, they can be enabled in the APP Settings > Switch Settings.


6. How to use the touch button

The wristband is touch button, please do not press it again; after entering any function interface, click the touch button again to enter the next interface in turn.

1. Main screen bright screen start: In the of screen state, long press the bottom of the bracelet, you can brighten the screen.
2. Press and hold on the calorie interface for 3 seconds to select whether to clear all current data;
3. Raise the hand and turn the wrist to brighten the screen. In the off-screen state, raise the hand and turn the wrist 90° to the chest, and wake up the bracelet.


7. APP Login and Register

1. Open H Band;
2. Enter the email address or mobile phone number in the first column;
3. Enter the password in the second column;
4. Click to register;
5. Improve personal information, personal data, set goals, and then click "√" to complete the registration.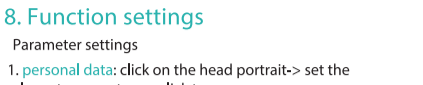

Parameter settings

1. **personal data**: click on the head portrait-> set the relevant parameters-> click to save
2. **My device**: Click the device name
- 2.1 **Message reminder**: click on the message reminder -> mark √ -> bind the phone -> exit
- 2.2 **Alarm settings**: click the alarm settings -> Edit -> Exit -> set the reminder time -> save -> click the switch
- 2.3 **Sedentariness reminder**: click the sedentariness reminder -> set the reminder time -> save -> click the switch
- 2.4 **Heart rate alarm**: click the heart rate alarm -> set the upper limit of the heart rate value -> save -> click the switch
- 2.5 **Palming bright screen**: click on the palming bright screen -> set the start and end time -> save -> click the switch
- 2.6 **Bind device switch**: click to bind device switch -> binding success -> click once again for unbinding
- 2.7 **Switch**: click on the switch settings -> wear detection and heart rate automatic monitoring is turned on by default
- 2.8 **Take pictures**: click to take pictures -> click on the bracelet camera icon to take pictures or wave the bracelet to take pictures
- 2.9 **Blood pressure private mode**: click on the blood pressure private mode -> set the upper limit of blood pressure data -> save -> click switch
- 2.10 **Reset the password**: click to reset the password -> enter the old password -> enter new password -> confirm new password -> reset password
- 2.11 **Firmware updating**: click firmware updating -> upgrade the firmware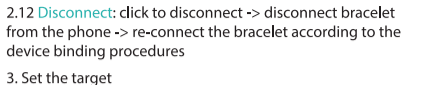

3. Set the target
- 3.1 **Movement target**: Click the movement target -> set the corresponding data -> save
- 3.2 **Sleep target**: Click the sleep target -> set sleep data -> save
- 3.3 **Unit Settings**: Click the unit settings -> set the unit system
- 3.4 **Feedback**: Click the feedback -> feedback description -> submit
- 3.5 **About Us**: Click About Us -> View the APP Version Number
- 3.6 **WeChat movement**: click the WeChat movement -> click on the binding
- 3.7 **Exit Login**: Click to log out -> repeat the login operation to login APP

4. Looking for mobile phone
When the bracelet APP is running at the background of the mobile phone, press the looking for mobile phone function. The phone will have music remind, click the looking for mobile icon interface once again to enter the next interface and the looking for mobile phone function is shut down automatically.

5. Heart rate test
- 5.1 Click the heart rate test interface, and the bracelet tests automatically;
- 5.2 APP test: open the APP to the heart rate interface, and click the subcentral test. The data can be uploaded to the WeChat Moments and WeChat friends, etc.

6. blood pressure test
- 6.1 Click to enter into the bracelet blood pressure test interface, and the bracelet tests blood pressure automatically;
- 6.2 APP test: open the APP to the blood pressure rate interface, and click the subcentral test. The data can be uploaded to the WeChat Moments and WeChat friends, etc.

7. Disconnect settings
7.1 After the bracelet is disconnected for exceeding the effective connection distance, once it enters into the connection distance again, open the phone APP, the mobile phone and bracelet will be connected automatically.
7.2 If you want to disconnect the connection between the bracelet and mobile phone completely, please open the phone APP, click the name of the bracelet in APP settings inside to enter the "My Equipment" page, click the "disconnect" at the bottom to disconnect them completely; for the ios phone, besides the above steps, it should also enter the phone "Settings" --- "Bluetooth", and click on the "exclamatory mark" behind the name of the bracelet to enter the "ignore this device" page, and click the "ignore the device" to remove the name of the bracelet.

8. Movement settings
Click on the movement to enter the movement interface, click to start the movement, and allow access to the location, to start recording the motion track. The APP will record the length of exercise, walking speed, the calories consumed, and mark the route on the map; click on the icon at the top right corner to share the record to QQ, WeChat and other social media.

9. Seventh, FAQ

- Q: Why can't I find the bracelet when matching?
A: 1. Please confirm you have opened Bluetooth of your mobile phone, while ensuring that the bracelet and mobile phone are within the range of 0.5 meters. After the completion of matching, the effective communication distance is 10 meters within a spacious space.
2. Please check whether the bracelet has electricity. Please contact us if it is still not resolved after charging.
3. Bracelet is beyond connection distance is no remind function effectively.

- Q: Why it can't connect to the Bluetooth device sometimes?
A: Some phones have abnormalities for bluetooth. Tru to re-start the phone Bluetooth or restart the phone, so that it can be connected normally.

- Q: Why can't appear blood pressure/heart rate manual test synchronization to the APP?
A: APP (H Band) blood pressure/heart rate manual monitoring data is not synchronized to the mobile phone above, can only be synchronized APP set automatic monitoring data to the H Band above view.

10. Basic parameters

Main control chip	Nordic 52832
Body weight	33g
Data transmission	Bluetooth4.0
Battery capacity	100mAh
Charging voltage	5V
Operating temperature	-10~50℃
Waterproof	IP67

Equipment requirements:
IOS 8.0 and above models;
Android 4.4 and above (Bluetooth 4.0) models.